CDS47V ChemSurf

3D Chemical Indicator

For disinfection processes control of environments and surfaces by Hydrogen Peroxide Vapor.

Usage

3D chemical indicator for monitoring tests of disinfection of environments and surfaces by Hydrogen Peroxide Vapor.

Conditions: H₂O₂ solution concentration ≥ 35%, temperature: 10-30°C, relative humidity: 30-80%.

Applicable regulation

Designed under Quality Management Systems standards ISO 13485:2016/NS-EN ISO 13485:2016.

Authorization

ANMAT (Argentinean National Administration of Drugs, Food and Medical Devices) PM 1614-4.

Classification

Class 1, according to risk (ANMAT).

Characteristics

3D Chemical Indicator with pyramidal shape.

Size: 10.5 cm side and 9.0 cm high (approximate).

Initial color: **red**

Final color: **yellow** (should change homogeneously on all exposed faces).

In its base it has a graph to mark the position of both the equipment and the ChemSurf indicator in the room to be disinfected. It also has a record chart to complete the date, operator and result.

Seals and supports are included that make it possible to suspend the indicator.

100 % Toxic Heavy Metals free.

To interpret results visually, use the reference found in the indicator.

Environmental conditions during manufacture

T = 15-30 °C, RH = 30-80 %.

Storage conditions

T = 10-30 °C, RH = 30-80 %, keep out of direct light in the original packing.

Transport conditions

Storage conditions should be strictly followed. Transport in closed and reinforced boxes in order to avoid damages. The transport of this product does not represent a risk for health.

Shelf life

5 years.

Packing

100 units, 5 seals and 5 supports per packing.

Documentation table.

Packing information: product code and description, process for intended use, presentation, manufacturer information and storage conditions.

Labelling

On product: product code and description, process for intended use, color change, batch number, expiration date, record chart and datamatrix code.

On product's packing: product code and description, process for intended use, color change, batch number, manufacture and expiration date, barcode and datamatrix code.

Possible target markets

Healthcare, Food, Pharmaceutical and Medical industries.

Other important information

Read product's instructions for use thoroughly before use.

Precautions

Do not store the product near sterilizing or disinfecting agents.

Do not expose this product to Steam, Dry Heat or any sterilization processes other than H₂O₂.

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